*No. 1 choice
of hospitals
& mothers*

Lactina™

- EN** Instructions for use
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing

BPA
free

EN Parts List XIV

DE Teilaliste XIV

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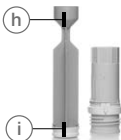
(e)

(h)

(i)

(m)

(o)

(n)

(a)

(b)

(c)

(d)

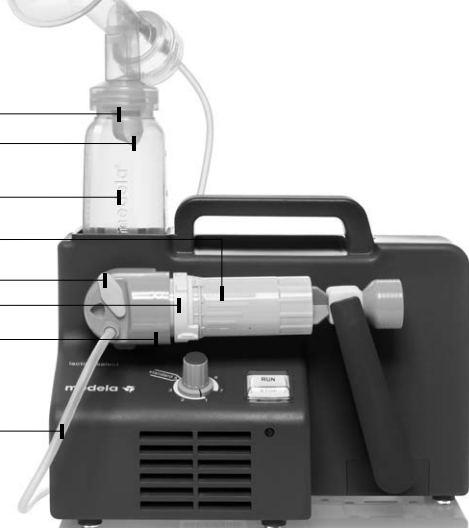
(g)

(l)

(j)

(k)

(f)

EN	Instructions for use	4
DE	Gebrauchsanweisung	18
FR	Mode d'emploi	32
IT	Istruzioni per l'uso	46
NL	Gebruiksaanwijzing	60
	EMC	74

I Lactina™ Breastpump

Mother's milk is the best nutrition for babies. Lactina gives you and your baby optimal support during the breastfeeding period. The attractive breastpump for hospitals and home rental is especially suitable for long-term and frequent pumping and encourages milk production in the long term, too.

II Indications For Use

The Lactina Breastpump is a powered breastpump:

- To be used by lactating women to express and collect milk from their breast.
- To initiate and maintain a milk supply as well as collect breastmilk at work or during other absences from the baby.
- To initiate and maintain a milk supply if direct breastfeeding is not possible.
- To relieve engorgement.
- For use in hospitals or home rental.

III Important Safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

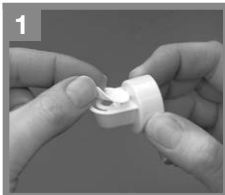
Read all instructions before use.

- *Use the product only for its intended use as described in this instruction manual.*
- *Only use the power cord that comes with the Lactina Breastpump.*
- *Make sure the voltage of the device is compatible with the power source.*
- *Always unplug electrical devices immediately after using.*
- *The product should never be left unattended when plugged into a power source.*
- *Separation from the mains is only assured through the disconnection of the power cord from the wall socket outlet.*
- *Keep the power cord away from heated surfaces.*
- *Never operate an electrical device if it has a damaged power cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.*
- *Do not use outdoors.*
- *Keep appliance dry and do not place in direct sunlight.*
- *Do not get the product wet.*
- *Do not use while bathing or showering.*
- *Do not place in or drop pump unit into water or other liquid.*
- *Do not spray or pour liquid directly onto the pump.*

- Do not store or use this product where it can fall or be pulled into a tub or sink.
- Do not reach for any electrical device if it has fallen into water. Unplug immediately.
- **Use Medela original accessories only.**
- Check pump kit components for wear or damage.
- The pump kit is a personal care item and is not to be shared between mothers without proper sterilization to prevent the risk of cross-contamination.
- Pump only with the motor in an upright position.
- Do not use while sleeping or overly drowsy.
- Supervision is necessary when this product is used in the vicinity of children.
- If problems or pain occur, consult your lactation consultant or physician.
- Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.
- Portable and mobile radio frequency communications equipment can affect the Lactina Breastpump.
- The Lactina Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.
- There are no user-serviceable parts inside the Lactina. Do not attempt to repair the Lactina Breastpump yourself.

SAVE THIS INSTRUCTION MANUAL

IV Assembly Instructions

b + c

Snap membrane onto valve until membrane lies completely flat.

a1 a2

+

b c

Push assembled valve and membrane onto breastshield. Ensure sidewise position of valve as indicated on picture.

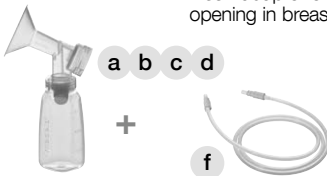
IV Assembly Instructions

3


Screw bottle into breastshield.

4

Insert adapter end of tubing into opening in breastshield.

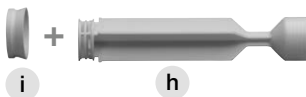5

Slip rubber seal onto piston until it snaps into place - the larger side faces the handle. To ensure proper seal, hold rubber seal firmly and twist piston.

6

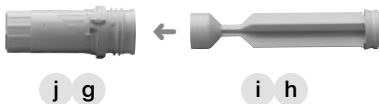
Slide vacuum regulator ring onto cylinder.

7

Slide piston into cylinder.


8

Snap port cap onto cylinder holder.


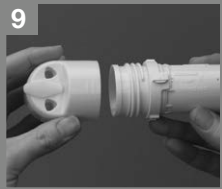
9

Screw piston/cylinder assembly tightly into cylinder holder.

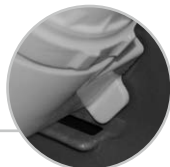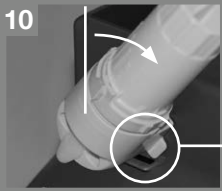
10

Insert plug at the back side of the cylinder holder into the hole on the front side of the Lactina. Rotate until tab of cylinder holder is securely placed in slit of Lactina.

IV Assembly Instructions

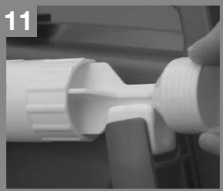11

Push piston into rubber clamp of Lactina pumping arm until it snaps in.

12

Push adapter end of tubing into opening on cylinder holder.

If single pumping, insert port cap into the unused opening of cylinder holder.

13

If double pumping, insert port cap into the hole in the middle of cylinder holder.

V Electrical Operation

Plug the power cord into a standard electrical outlet.

VI Pumping

Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.

Center your breastshield over your nipple.

- Do not tilt or overfill bottles when pumping.
- Take care not to kink tubing while pumping.

Set the vacuum regulator to minimum and switch on the breastpump.

◇ On

▽ Off

Adjust the vacuum to your **Maximum Comfort Vacuum™**. This is the highest vacuum you can use and still feel comfortable (different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

Research has shown that you will pump more efficiently - meaning get more milk in less time - when pumping at Maximum Comfort Vacuum™.

Once finished pumping, switch off the breastpump, unplug it, and disassemble the pump kit items. Close the bottle and store it for later use.

Use bottle stand to prevent tip over.

VI Pumping

Lactina Select only

The Lactina Select features an additional knob to adjust the pumping speed. Choose a fast pumping rhythm to start your milk flowing (setting 7 corresponds to highest speed level). Once your milk starts to flow or you sense milk ejection you may adjust the rhythm by decreasing speed slightly (e.g. setting 4).

VII Breastshield Fitting

Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

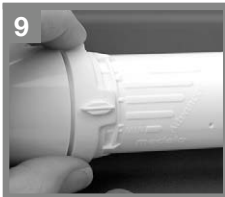
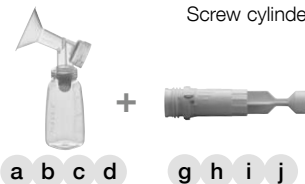
- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

Medela PersonalFit™ Breastshield: Available in five sizes - because one size does not fit all.

VIII Manual Pumping

Follow steps 1–7 (excluding step 4) on page 3/4/5

Set the vacuum regulator to minimum and start pumping. Adjust the vacuum to your **Maximum Comfort Vacuum™**. This is the highest vacuum you can use and still feel comfortable (different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

IX Cleaning and Hygiene

Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Disassemble all parts prior to cleaning. Especially be sure to remove the yellow valve head from the breastshield and separate the white valve membrane from the valve head.

IX Cleaning and Hygiene

Cleaning at home

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Prior to first use and once daily

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+m+n+o).
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

Alternatively, the disassembled parts can be sanitized in top rack of dishwasher, with the Medela Quick Clean microwave bags.

Note: Food colorings may discolor components which will not affect performance.

After each use

- Disassemble and handwash all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+m+n+o) in warm, soapy water.
- Rinse in clear water.
- Dry parts with a clean towel.

Cleaning in hospitals

The following cleaning instructions are general recommendations and may individually be adjusted based on the hospital's specific directives and cleaning practices.

Important

It is recommended to provide each mother an own personal pump set and to clean, disinfect and sterilize (e.g. autoclave) all components after each use or, latest each time they are handed over to the next mother. Use by more than one mother without adequate sterilization may pose a health risk (e.g. risk of cross-contamination) and voids the warranty.

Cleaning

Disassemble and thoroughly wash all parts in warm, soapy water. Rinse in clear water.

Disinfecting

Disinfect all parts by washing in a thermic washing machine (at approx. 93° C).

Sterilizing

Prior sterilizing visually check parts if clean and intact. Sterilize all parts by autoclaving at 134° C for 3 (or 18) minutes or at 125° C for 15–20 minutes. Do not stack parts or apply external pressure to avoid damages. Alternatively, the parts can be EO-sterilized (Ethylene Oxide).

Tubing care

Inspect the tubing after each pumping session for condensation and/or milk.

If condensation appears in the tubing

Let the pump run after you have completed pumping with tubing attached for 1–2 minutes or until dry.

If milk gets into the tubing

If milk accidentally gets into the tubing turn off the pump and unplug from power source. Remove and clean the tubing according to the instructions «Prior to first use and once daily». Shake water droplets out and hang to air-dry. For faster drying, attach the tubing to pump and run for 1–2 minutes or until dry.

Hints

After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

Hygiene

- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Avoid touching the inside of bottles and lids.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol which can dry the skin and cause cracked nipples.

IX Cleaning and Hygiene

Breastpump care

Unplug breastpump prior to cleaning

At home

Daily or when spills occur

In hospitals

Immediately when spills occur

In between each mother and/or on a regular basis, e.g. daily

- Use a damp (not wet) cloth to wipe over the breastpump. Optionally, soapy water or a non-abrasive detergent may be used.
- Do not spray or pour liquid directly onto the pump.
- Do not immerse the pump unit in water.

X Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- *When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.*
- *Label bottles/bags with the date of pumping.*
- *Do not store breastmilk in the door of the refrigerator.*
- *In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).*
- *Gently shake the bottle/bag to blend any fat that has separated.*
- *Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).*

XI Troubleshooting

If the motor is not running

Check

- mains power is present
- the device is switched on

If there is low or no suction

Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See «Assembly Instructions».
- the ends of the tubing are fit securely into the back of the breastshield and into the opening on cylinder holder.
- the rubber seal is correctly attached onto piston.
- the piston/cylinder is correctly attached onto cylinder holder or breastshield.
- the cylinder holder including piston/cylinder is assembled properly onto the Lactina.
- If single pumping, make sure the port cap is inserted into the unused opening of cylinder holder.

Take care not to kink or pinch tubing.

If there is an overflow

If milk accidentally gets into the tubing

- Turn off the pump and unplug from power source.
- Use a damp (not wet) cloth to wipe the pump.
- Refer to cleaning instructions «Tubing care and Breastpump care».
- If necessary clean the pump kit.
- When components are completely dry, reassemble breastpump and kit.

XII Warranty

3-year warranty for the unit from the date of purchase, excluding pump set and accessories. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons. Routine checks and servicing may only be carried out by Medela authorized personnel.

XIII Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment. Improper disposal may have harmful effects in the environment and on public health.

XIV Parts List

Article numbers

a1	Breastshield (24 mm)	810.7084
a2	Connector	810.7088
b	Valve head	810.0475
c	Valve membrane	820.7013
d	Bottle (150 ml)	811.0020
e	Bottle stand	810.0462
f	Silicone tubing (105 cm)	800.0828
g	Cylinder	810.0717
h	Piston	810.0494
i	Rubber seal	200.2575
j	Vacuum regulator ring	200.2775
k	Cylinder holder	810.0668
l	Port cap	820.0135
m	Cap	810.0474
n	Lid	810.7062
o	Disc	810.0473

Accessories (sold separately)

- One-Day Pump Set for a limited number of uses (max. 24 hours or 8 pumping sessions) 008.0179/008.0180/008.0181
- Disposable bottles (80 ml/150 ml/250 ml) for one-time use only 200.2957/200.2956/200.4802
- Car adapter 928.7001

*Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.
Subject to change.
Content may vary from picture.*

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Electromagnetic Compatibility (EMC, IEC 60601-1-2:2001)

IEC 60601-1-2, Table 201

Electromagnetic Emissions		
The Lactina breastpump is intended for use in the electromagnetic environment specified below. The customer or the user of the Lactina breastpump should assure that it is used in such environment.		
Emission Tests	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Lactina breastpump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Lactina breastpump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

Warning – The use of accessories and cables other than those specified may result in increased emissions or decreased immunity of the Lactina breastpump.

The Lactina breastpump should not be used adjacent to or stacked with other equipment.

If adjacent or stacked use is necessary, the Lactina breastpump should be observed to verify normal operation in the configuration in which it will be used.

The Lactina breastpump was found to have no essential performance by a risk analysis and thus, was not tested for immunity to electromagnetic disturbances (IEC60601-1-2:2001, 6.8.3.201 i)

EN Technical specifications
 DE Technische Daten
 FR Caractéristiques techniques
 IT Caratteristiche tecniche
 NL Technische gegevens

vacuum (approx.)
 -100 ... -240 mmHg
 -13.33 ... -32 kPa
 Lactina Electric Plus 50 cpm
 Lactina Select 40 - 60 cpm

100 -240 V
 50 / 60 Hz
 50 VA ~ alternating current

T 1.25 A / 6.3 x 32 / 250 V

Medela Art. Nr. 928.7001
 Car adapter / Autoadapterkabel
 Câble de branchement sur
 allume-cigares / Adattatore per
 automobile / Auto-adapter

12 V DC
 16
 T 1.25 A --- direct current

kg
 2.4 kg

Operation
 Betrieb
 En Service
 Utilizzo
 Gebruik

Transport / Storage
 Transport / Lagerung
 Transport / Stockage
 Transport / Stoccaggio
 Transport / Opslag

Operation
 Betrieb
 En Service
 Utilizzo
 Gebruik

Transport / Storage
 Transport / Lagerung
 Transport / Stockage
 Transport / Stoccaggio
 Transport / Opslag

Operation / Transport / Storage
 Betrieb / Transport / Lagerung
 En Service / Transport / Stockage
 Utilizzo / Transport / Stoccaggio
 Gbruikt / Transport / Opslag

Protection class II
 Type B

FDA (GMP)
 ISO 9001
 ISO 13485
 CE (93/42/EEC)

Please see accompanying papers /
 Begleitpapiere beachten / Respecter les
 consignes des documents d'accompagnement /
 Consultare la documentazione allegata /
 Zie bijgevoegde documenten

Must not be disposed of together with house-
 hold refuse / Darf nicht zusammen mit Haus-
 müll entsorgt werden / Ne doit pas être éliminé
 avec les ordures ménagères / Non deve essere
 smaltito con i rifiuti domestici / Mag niet samen
 met huisvuil weggegooid worden

Tested according to
 IEC 60601-1/A2:1995

www.medela.com

Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

International Sales

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 (0)41 769 51 51
Fax +41 (0)41 769 51 00
ism@medela.ch
www.medela.com

France

Medela France Sarl
14, rue de la Butte Cordière
91154 Etampes cedex
France
Phone +33 (0)1 69 16 10 30
Fax +33 (0)1 69 16 10 32
info@medela.fr
www.medela.fr

Germany & Austria

**Medela Medizintechnik
GmbH & Co. Handels KG**
Postfach 1148
85378 Eching
Germany
Phone +49 (0)89 31 97 59-0
Fax +49 (0)89 31 97 59 99
info@medela.de
www.medela.de

Italy

Medela Italia Srl
Via Turrini, 19 - Loc. Bargellino
40012 Calderara di Reno (BO)
Italy
Phone +39 051 72 76 88
Fax +39 051 72 76 89
info@medela.it
www.medela.it

Netherlands & Belgium

Medela Benelux BV
Uilenwaard 31
5236 WB 's-Hertogenbosch
Netherlands
Phone +31 73 690 40 40
Fax +31 73 690 40 44
info@medela.nl, info@medela.be
www.medela.nl, www.medela.be

Switzerland

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 (0)848 633 352
Fax +41 (0)41 769 51 00
contact@medela.ch
www.medela.ch

United Kingdom

Medela UK Ltd.
Huntsman Drive
Northbank Industrial Park
Irlam, Manchester M44 5EG
UK
Phone +44 (0)161 776 0400
Fax +44 (0)161 776 0444
info@medela.co.uk
www.medela.co.uk